



# INSTRUCTIONS

## ⚠️ WHEELCHAIR USE

- **SEATBELT MUST BE USED AT ALL TIMES.**
- **CAPSULE SEAT MUST BE IN DOWN POSITION.**
- **PUSH WHEELCHAIR INTO CAPSULE AS FAR BACK AS POSSIBLE.**
- **APPLY HANDBRAKE OR TURN OFF POWER.**
- **APPLY WHEEL LOCKING CHAIN.**
- **FIT AND FASTEN SEATBELT.**

## ⚠️ CAPSULE SEAT USE

- **SEATBELT MUST BE USED AT ALL TIMES.**
- **1 LIFT SEAT UP AND OUT OF THE WAY.**
- **2 LIFT SUPPORT LEG INTO POSITION.**
- **3 LOWER SEAT INTO LOCK POSITION ON SUPPORT LEG.**
- **4 SEAT USER, FIT AND FASTEN SEATBELT.**

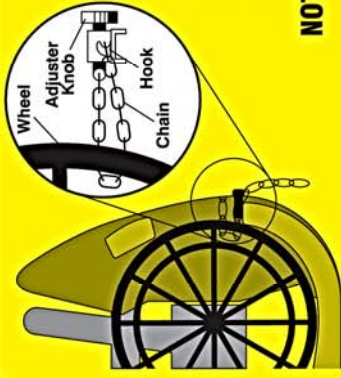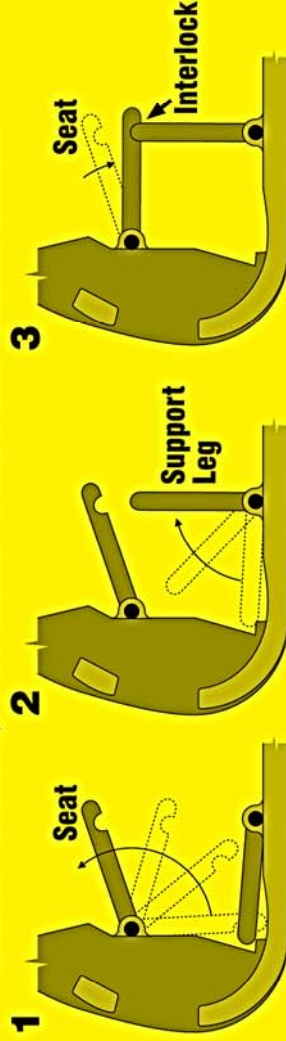


### WHEEL LOCKING DEVICE

1. Adjustor knob to be wound out by turning anti-clockwise.
2. Pull chain through wheel firmly & secure over hook then tighten adjustor knob.
3. Tighten by turning adjustor knob clockwise.
4. Check wheelchair for stability.

**NOTE:** Some wheelchairs may require chain to be fixed through main sub-frame.

### SEATBELT IN BOX REPLACE AFTER USE

- 1 Remove seatbelt from compartment.
- 2 Attach seatbelt to anchor points and adjust.

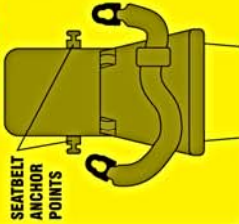

### PREVENT BACK INJURY



**Bend Knees To Lift**